FOR ENERGY EFFICIENT INNOVATIONS

**THINK ON.**

[www.onsemi.com](http://www.onsemi.com)

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TABLE 1: SUPPORTED COMMUNICATION INTERFACES AND PROTOCOLS

Interface	RS232	I <sup>2</sup> C
HI-PRO	X	X
Microcard	X	
DSP Programmer 1	X	
DSP Programmer 2	X	
DSP Programmer 3	X	X
NOAHlink	X	X

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[ARK Downloads](#)

[Release Highlights](#)

ARK is a set of software building blocks that can help reduce the time it takes to develop hearing instrument fitting software. ARKonline™ is an efficient web-based tool for creating product libraries and keeping them organized.

Libraries can be created using custom or standard maps, microphone models, and receiver models. Libraries previously created by the user are stored in ARKonline for easy editing or downloading.

### Note:

Downloading any ARKonline software package from ON Semiconductor requires a login name, password and the proper download privileges. A login name and password can be created by registering with ARKonline™ [here](#).

\* ARKonline™ is a trademark of Semiconductor Components Industries, LLC.

# Pre-configured 开发

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- [Home](#)
- [Downloads](#)
- [Documents](#)
- [Library Manager](#)
- [Private Downloads](#)
- [Preferences](#)

You are logged in as snow

### Software Updates

**ARKbase 6.6.2, ARKsdk 6.6.2** are now available a

[ARK Downloads](#)

[Release Highlights](#)

ARK is a set of software h

ON Semiconductor®

## DOWNLOADS

[Home](#)

E-mail

[has.software@onsemi.com](mailto:has.software@onsemi.com)

You are logged in as

Below is a list of our most recent software packages available for download. For a more thorough description of the software package, click on the name of the software package in question.

### [ARKbase](#)

ARKbase contains all ARK components and several applications for working with ON Semiconductor products

### [ARKbase CE](#)

ARKbase CE contains all ARK components and several applications for working with ON Semiconductor products on Windows CE-based devices.

### [ARKsdk](#)

ARKsdk is for software developers. It contains source code for most of the applications included in ARKbase.

### [ARKfit](#)

ARKfit consists of source code for familiarizing software developers with ON Semiconductor's auto-fitting routines.

### [SOUNDFIT™](#)

SOUNDFIT™ is ON Semiconductor's fitting software application.

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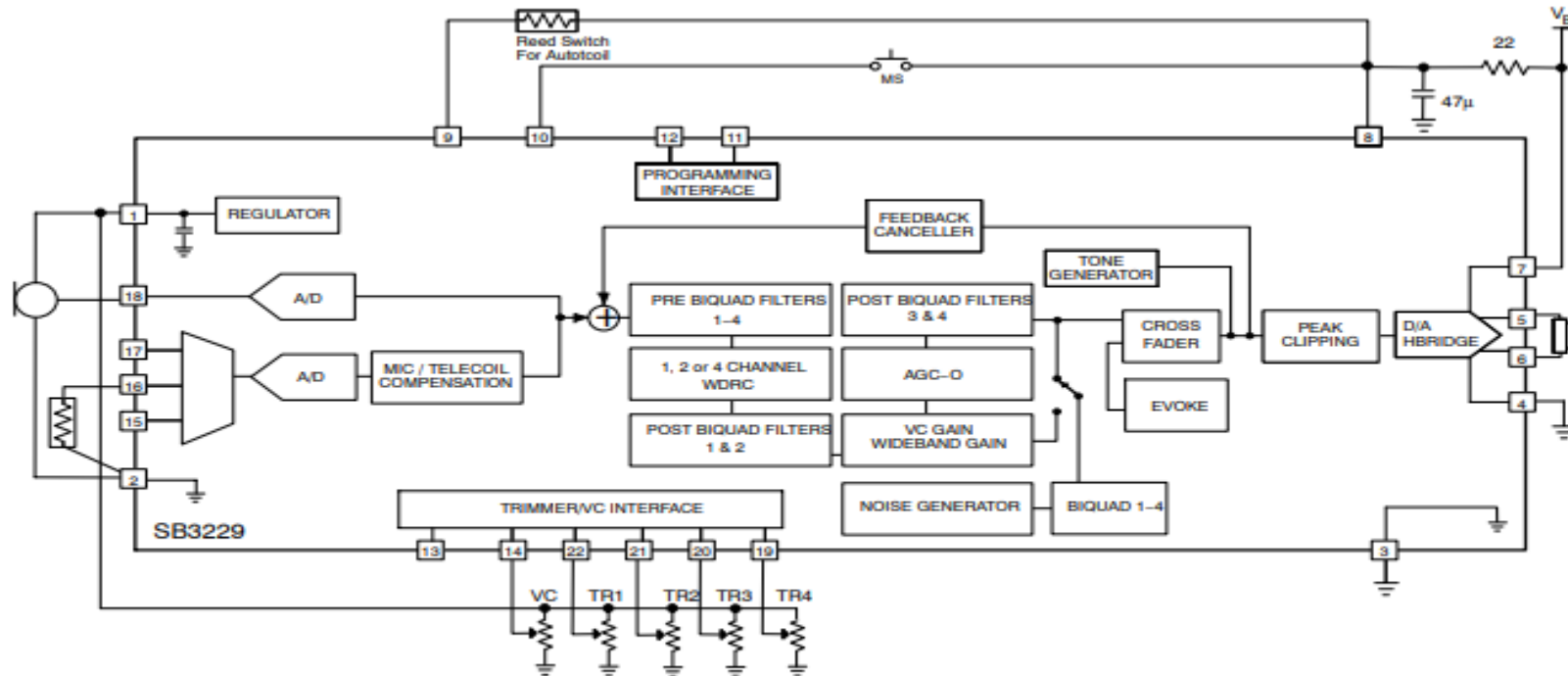
Note: All resistors in ohms and all capacitors in farads, unless otherwise stated.

Figure 4. Typical Trimmer Application Circuit

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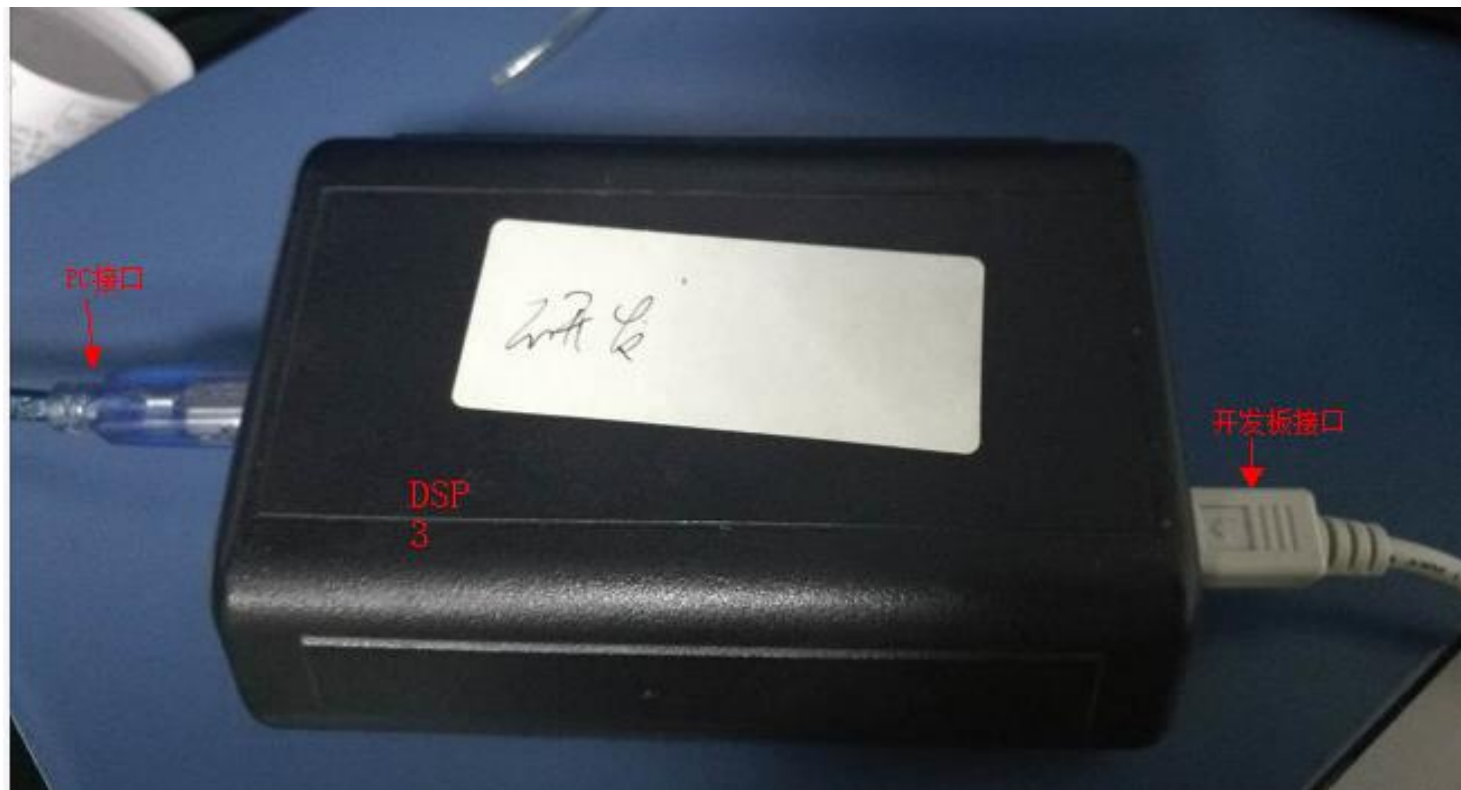
连接 Hi-pro 与 Fonix8000 到 PC

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Ports (COM & LPT)

Intel(R) Active Management Technology - SOL (COM3)

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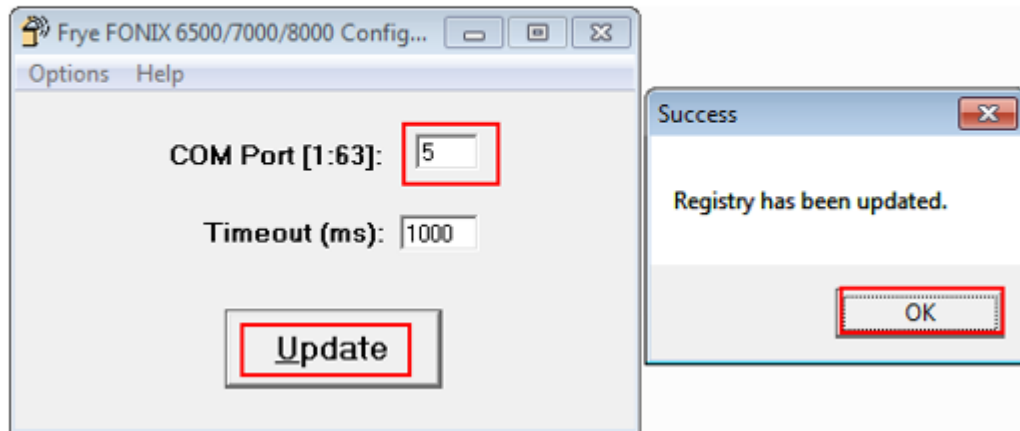
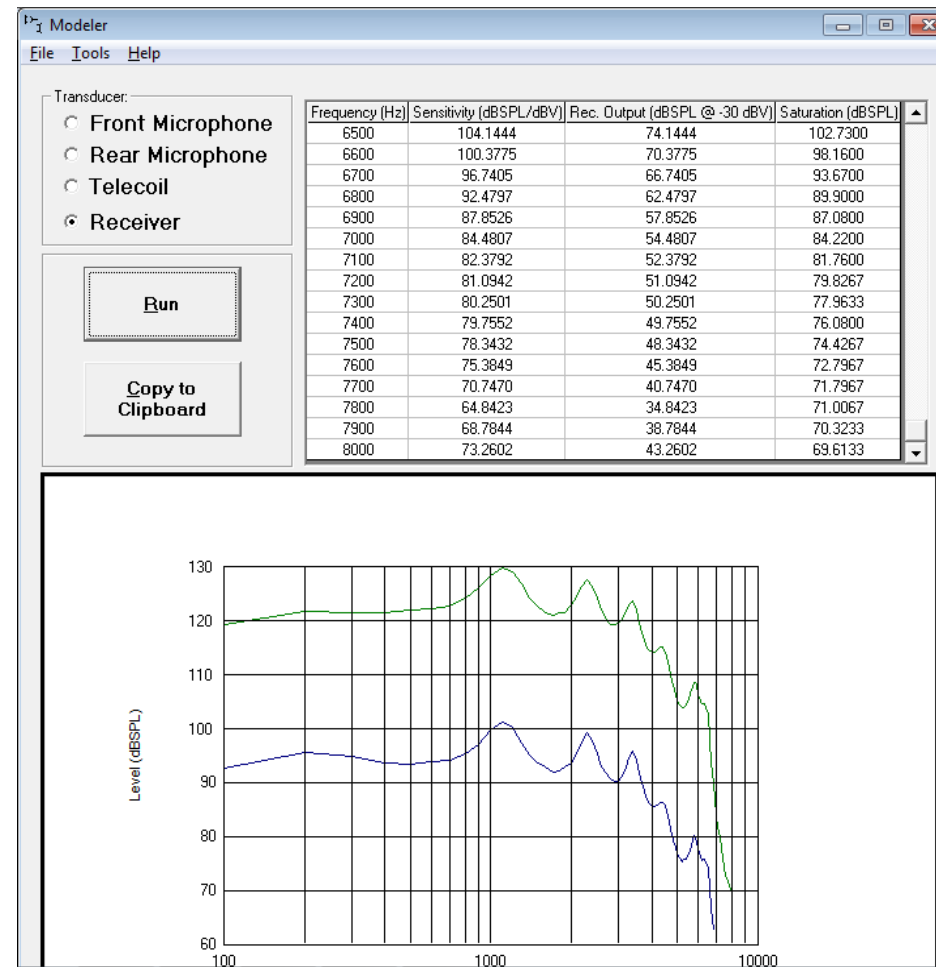
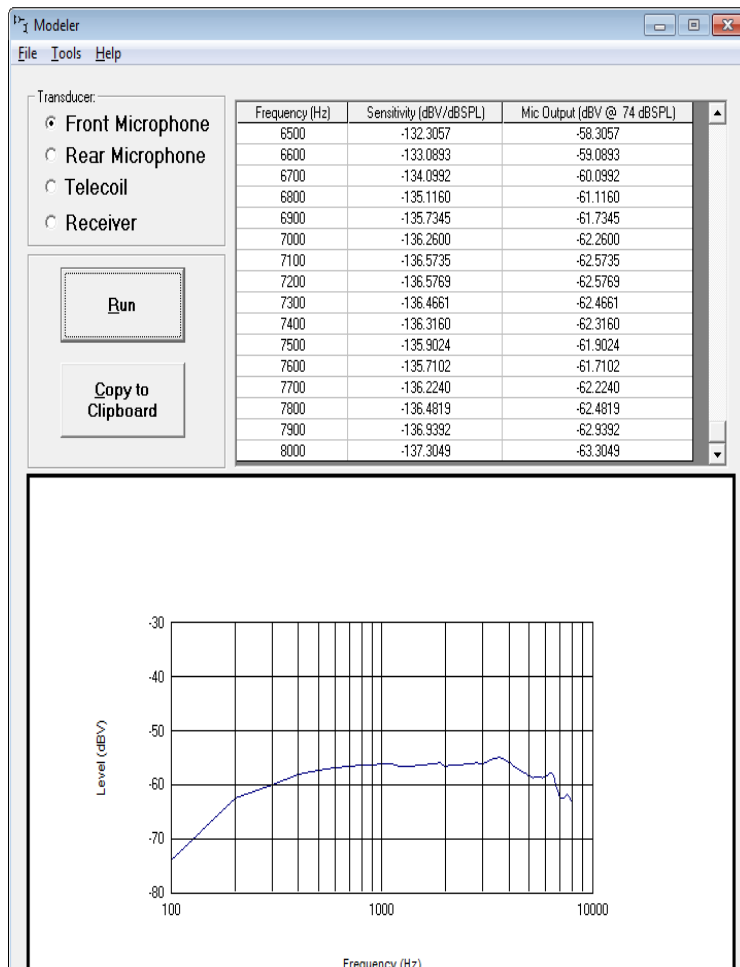
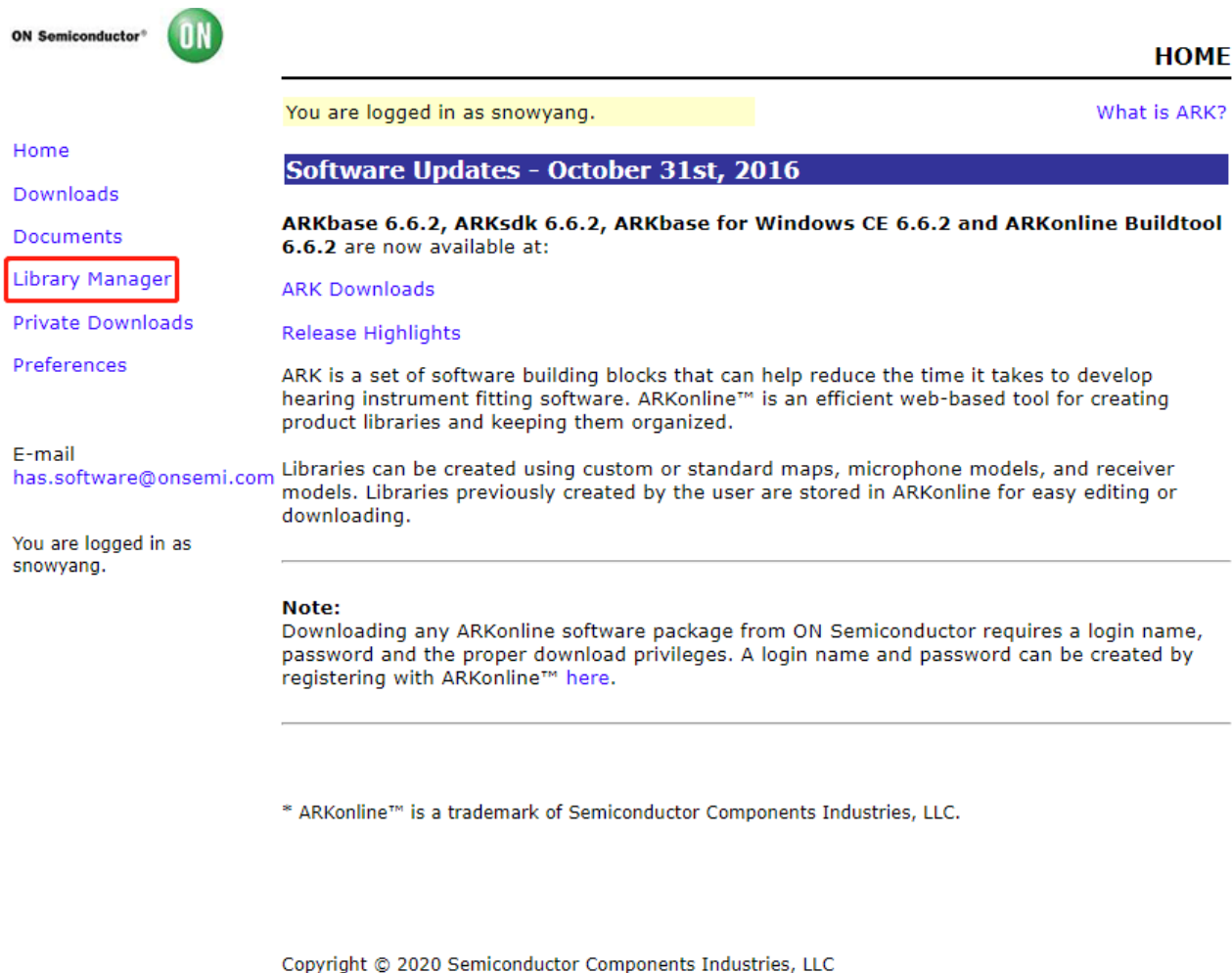
# Pre-configured 开发

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The screenshot shows the ON Semiconductor ARKonline website. At the top left is the ON Semiconductor logo. On the right is a 'HOME' link. Below the logo is a navigation menu with links for Home, Downloads, Documents, Library Manager (highlighted with a red box), Private Downloads, and Preferences. Below the menu is the user's email address, has.software@onsemi.com, and a confirmation that the user is logged in as snowyang. The main content area features a blue header for 'Software Updates - October 31st, 2016'. Below this, it announces that ARKbase 6.6.2, ARKsdk 6.6.2, ARKbase for Windows CE 6.6.2, and ARKonline Buildtool 6.6.2 are now available. There are links for ARK Downloads and Release Highlights. A paragraph explains that ARK is a set of software building blocks for developing hearing instrument fitting software. A 'Note' section states that downloading any ARKonline software package requires a login name, password, and proper download privileges, and provides a link to register. At the bottom, there is a copyright notice for 2020 Semiconductor Components Industries, LLC.

ON Semiconductor® 

HOME

You are logged in as snowyang. [What is ARK?](#)

**Software Updates - October 31st, 2016**

**ARKbase 6.6.2, ARKsdk 6.6.2, ARKbase for Windows CE 6.6.2 and ARKonline Buildtool 6.6.2** are now available at:

[ARK Downloads](#)

[Release Highlights](#)

ARK is a set of software building blocks that can help reduce the time it takes to develop hearing instrument fitting software. ARKonline™ is an efficient web-based tool for creating product libraries and keeping them organized.

E-mail [has.software@onsemi.com](mailto:has.software@onsemi.com)

You are logged in as snowyang.

**Note:**  
Downloading any ARKonline software package from ON Semiconductor requires a login name, password and the proper download privileges. A login name and password can be created by registering with ARKonline™ [here](#).

\* ARKonline™ is a trademark of Semiconductor Components Industries, LLC.

Copyright © 2020 Semiconductor Components Industries, LLC

## • 导入microphone与receiver曲线

ON Semiconductor® 

### MICROPHONES

Microphone Search:

[Add New Microphone](#)

Unsaved Microphones  Saved Microphones

#### onsemi Microphones

Title	Version	Manufacturer	App.	
EM24446_C36	1.0.0	Knowles	ITC	<a href="#">View</a>
Above is being edited by: ffvntn				
test1	1.0.0	Knowles	CIC	<a href="#">View</a>
Above is being edited by: ffvntn				
S6374	1.0.0	Sonion	CIC	<a href="#">View</a>
Above is being edited by: ffvntn				
TM2673	1.0.0	Sonion	ITC	<a href="#">View</a> <a href="#">Copy</a> <a href="#">Edit</a>
Above is owned by: ffvntn				
S6374_5	1.0.0	Sonion	ITC	<a href="#">View</a> <a href="#">Copy</a> <a href="#">Edit</a>
Above is owned by: ffvntn				
TM26403	1.0.0	Sonion	ITC	<a href="#">View</a> <a href="#">Copy</a> <a href="#">Edit</a>
Above is owned by: ffvntn				
S6374_pro	1.0.0	Sonion	CIC	<a href="#">View</a> <a href="#">Copy</a> <a href="#">Edit</a>
Above is owned by: ffvntn				

#### Public Microphones

Title	Version	Manufacturer	App.	
EM-4346CX	1.0.0	Knowles	ITE	<a href="#">View</a> <a href="#">Copy</a>
M_9646GX	1.0.0	Sonion	ITE	<a href="#">View</a> <a href="#">Copy</a>

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Allow other people in my company to view this item.

Allow other people in my company to edit this item.

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Receiver Search:

[Add New Receiver](#)

Unsaved Receivers  Saved Receivers

#### My Receivers

Title	Version	Manufacturer	Type	App.	
Rec_for_pre	1.0.0	Knowles	Zero-Biased	BTE	<a href="#">View Copy</a> <a href="#">Edit</a> <a href="#">Delete</a>

onsemi Receivers

[Home](#)

[Library Manager](#)

[Libraries](#)

[Microphones](#)

[Receivers](#)

[Maps](#)

E-mail

[has.software@onsemi.com](mailto:has.software@onsemi.com)

SA3229\_S01 Wizard

[Previous](#)

[Next](#)

[Save](#)

[Abort](#)

### System Features 系统功能设置

#### Sampling Frequency 系统采样率

- Set to 16 kHz
- Set to 32 kHz

#### Startup Program 启动程序

- Shallow reset mode.
- Deep reset mode.
- Advanced reset mode (startup with shallow reset, switch to transient reboot when low battery voltage detected).
- Enable Hybrid Authentication 芯片认证

#### Note:

The Sampling Frequency selection alters the recommended Feedback Canceller Delay, if the Feedback Canceller is enabled. Please ensure that this value is set correctly on the Advanced Features page.

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[Create New Library](#)

**Current Buildtool Version:** 6.6.2

Unsaved Libraries    Saved Libraries

**My Libraries**

Title	DLL	Source	Status	
SB_SA3229_test_library			Pending	<a href="#">View</a> <a href="#">Copy</a> <a href="#">Edit</a> <a href="#">Delete</a>

**LIBRARIES**

---

Library Search:

[Create New Library](#)

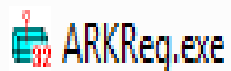
**Current Buildtool Version:** 6.6.2

Unsaved Libraries    Saved Libraries

**My Libraries**

Title	DLL	Source	Status	
SB_SA3229_test_library	<a href="#">Download</a>		Built Using v6.6.2	<a href="#">View</a> <a href="#">Copy</a> <a href="#">Edit</a> <a href="#">Delete</a>

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The screenshot shows the JHER software interface. At the top, there are two ear configuration panels for '右耳' (Right Ear) and '左耳' (Left Ear), each showing '6 memories, 16 Channel ITE/ITC 119' and a 'Memory 0' dropdown with a '保存' (Save) button. Below this is a '连接助听器' (Connect Hearing Aid) button. A red arrow points to this button with the text '点这里连接助听器, 读取助听器的参数'. To the right of the arrow, another red text block says '助听器连接状态, 如果有助听器连接, 按钮会红色/蓝色, 应用程序可选'. On the left side, there is a vertical menu with icons and labels: '用户管理', '助听器型号', '自动验配', '基本验配', '详细验配', '个性设置', '程序设置', '完成验配', and '验配助手'. At the bottom, there are three status bars: '状态栏1' (Status Bar 1), '状态栏2:主要显示验配工程中详细的错误信息' (Status Bar 2: Mainly displays detailed error information in the fitting process), and '当前验配的用户' (Current fitting user) with the text '您还没有选择用户' (You have not selected a user).

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JHER

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右耳 4 channel, Constant HLGAIN Memory 0 保存

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JHER

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右耳  
4 channel, Constant HLGAIN  
Memory 0 保存

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